


Rosie SmartScale

Digital Chair Scale Set Up

- **Tools Required:** Knife & Help
- **In Svc Requirement: 2 shifts (am/pm)**
 - Ask DON or SDC to assist w/ times & attendance
 - Send In Service Invitation
- **Open Box, Remove Packaging**
- **Get Help! 2 people** required to lift chair out of box
- **No Assembly Required**
 - Our scale does **NOT** include footrest
- **Place scale on floor (carefully w/ help)**
 - Ensure **wheels** operate smoothly
 - Ensure both **arms** articulate freely
- **Install Power Cord** (note the Charger “nest”)
- **Attach Quick Start Guide**
- **TEST: RM to test equipment**
 - Take your own weight
 - Calculate your BMI

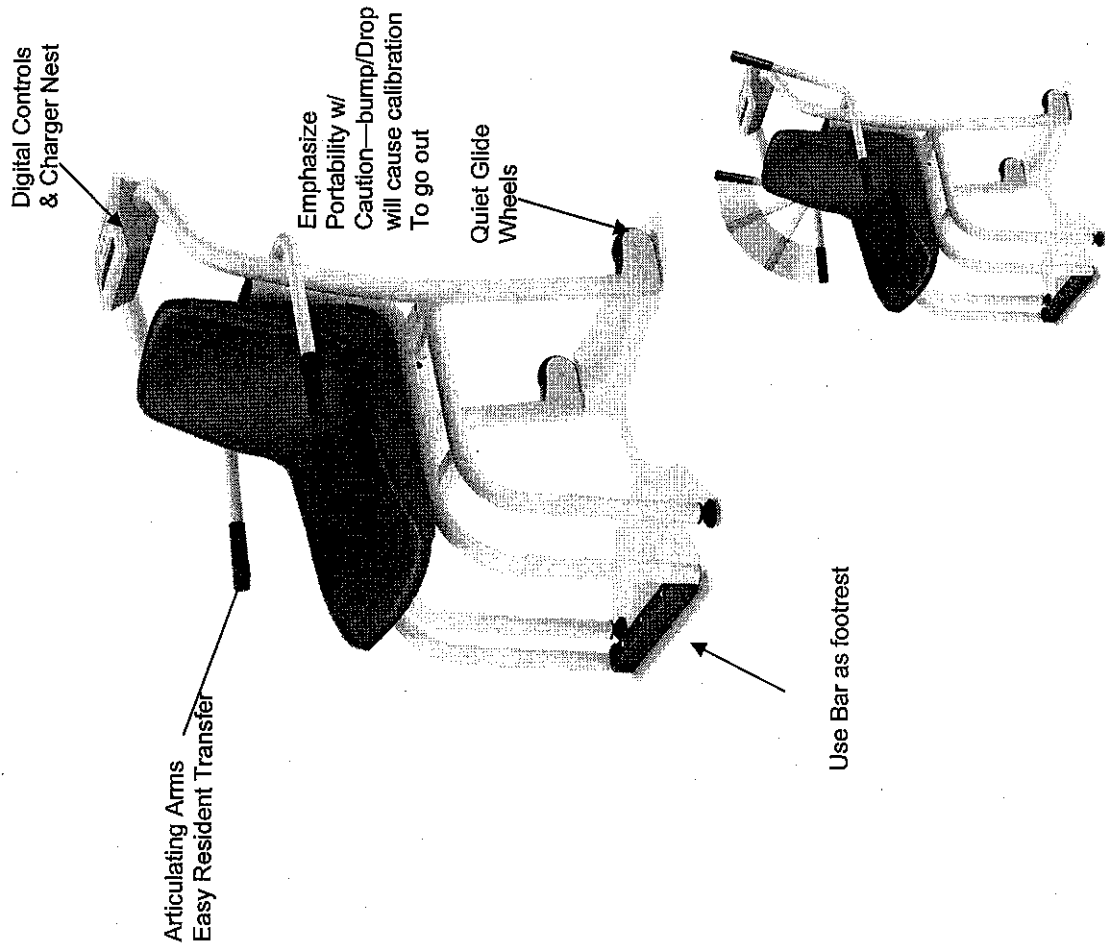

Rosie SmartScale: Chair Scale

Team Training

- **Tools:** DVD, Ops Manual; QuickStart Tag
- **Why Rosie Chair Scale?**
 - Capacity to 600 lbs
 - Independent swing arms for easy resident transfer
 - Quiet, heavy duty wheels for easy relocation to resident bedside
 - MotionTrap movement compensation assures accuracy
 - Easy, intuitive controls w/ BMI function
 - 3 Year Warranty (others offer 2 year)
 - RosieConnect Building Block
- **Set Up: Move Scale to resident location**
 - Downward leverage on back handlebar to raise front legs
 - Guide to desired location (no wall bumping!)
 - **GENTLY Lower front legs to floor in "level" location**
- **Weighing Resident:**
 - Turn scale on: Press On-Off key; 0.0 will appear
 - Lift Chair Arms to facilitate resident transfer to seat
 - Place residents feet on on bar between chair legs
 - Weight will appear in display (audible beep)
 - Press Reweigh to confirm weight
 - Initiate BMI calculation if desired, or
 - Assist resident to exit scale (lift arms)
 - Clear weight by pressing On-Off key or press & hold to turn off
- **Calculating Body Mass Index (BMI)**
 - Obtain weight
 - Press BMI Key (default height of 5 ft will appear)
 - Use Up/Down arrows to increase/decrease feet by 1 ft increment
 - Press BMI Key Again: inches will display w/ default of 7 inches
 - Use Up/Down arrows to increase/decrease by ½" increments
 - Press BMI Key again to confirm inches
 - Final Height will display: 5—0.75 = 5 ft, 7 ½ in.
 - Press BMI Key Again, Calculated BMI will appear in display
 - Press BMI Again to clear Display
- **Be GENTLE: Do NOT Drop/Bump: affects accuracy**

